

Note: Always scan the horse prior to injecting. If a microchip is found, call 800.434.2843 for information.



1. Locate the injection site. On the left side, halfway between the poll and withers, locate the nuchal ligament, just below the mane.



2. Prepare the site for aseptic injection.

3. Standard Injection: Inject the microchip directly into the prepared site, perpendicular to the neck. (You can also inject on an angle for younger horses.) Slowly depress the syringe plunger until it stops.

Apply pressure on the injection site as the needle is withdrawn.



4. Scan the horse.



5. Record the microchip number.



6. Enroll in HORSEtrac™ (800) 434-2843

